

## EMI in the Workplace Environment

Medical devices have become very sophisticated since the introduction of the first implantable pacemaker in the late 1950's. Similarly, technology of items found in the home and workplace environments have become very sophisticated to the point where interaction between some products and implanted devices can occur. Mechanical and electrical shielding designed into pacemakers and implantable cardioverter-defibrillators (ICDs), has, in most cases, enabled these medical devices to be immune to external electromagnetic interference (EMI). Most of the common home and workplace items that can generate EMI typically do not interfere with normal operation of implantable medical devices.

There are instances where equipment in a home, workplace or other environment may produce levels of electromagnetic interference capable of interfering with an implantable pacemaker or ICD. When interference is encountered, there are various device responses (some of which are listed below) that can occur and are usually temporary until the source of the EMI is removed or the patient moves outside the EMI field. If interference is encountered, once the patient is no longer exposed to the EMI field, normal device function usually returns. In the event that an EMI source strong enough to affect an implanted device is encountered, one or more of the potential responses listed in Table 1 below may occur.

**Table 1 – Potential device responses.**

<b>Response</b>	<b>Pacemakers</b>	<b>ICDs</b>
<i>A description of each is outlined in the bullets below.</i>		
Asynchronous pacing/ Magnet mode operation	Possible	Possible
Inhibition	Possible	Possible
Tracking up to programmed Maximum Rate	Possible	Possible
Inappropriate shocks/ATP	N/A	Possible
Inhibition of High Voltage/ATP therapy	N/A	Possible
Device Damage	Unlikely	Unlikely

- Asynchronous pacing due to electrical noise is a safety feature that forces pacing regardless of the patient's own rhythm. Magnet mode will also cause asynchronous pacing in response to a magnetic field in pacemakers.
- Inhibition of pacing (pauses in pacing) may occur at the onset of noise or with continued noise.
- In dual chamber devices noise detected on the atrial channel may result in temporary increased ventricular paced rates within the programmed limits.
- Noise could inhibit or trigger high voltage/Anti-Tachycardia Pacing (ATP) therapy.
- The presence of a static magnetic field may inhibit shock therapy.
- An external high voltage shock may result in device damage. This type of interference is not specified in the Field Strength Test Limits table below.

The susceptibility of an implanted device being affected by the environment or an external source of EMI is influenced by many factors including the patient body mass, the proximity and orientation to the EMI field as well as the EMI field strength. For patients who work in, or frequent locations that may expose them to high levels of electromagnetic interference, specific testing of an environment should be undertaken if

a medical professional deems there is sufficient risk to the patient based on the patient's medical history.

St. Jude Medical does not provide field testing of potential sources of interference. Patients who work at facilities where EMI sources may be present should consider advising their employer that they have an implanted medical device and that environmental testing may be in order. This type of assessment can be performed by a trained EMI testing consultant at the employer's request. Testing typically includes the work areas the employee would encounter, but may include the entire facility as the employer and tester deem necessary. A list of some potential EMI test consultants is provided to help facilitate this testing. St. Jude Medical does not recommend any particular consultant nor do we ensure the quality of the EMI testing performed. EMI field strengths encountered above those listed in Table 2 may cause device interference with an undesired device response.

**Table 2 - Field Strength Test Limits**

<b>EMI Source</b>	<b>Field type</b>	<b>Field strength limit</b>
Power Frequency (50/60 Hz)	E-field	6000 V/m
Modulated magnetic fields (50/60 Hz)	H-field	80 A/m
Static Magnetic Field	B-field	5 Gauss (0.5 mT)

St. Jude Medical pacemakers and ICDs are designed to meet the electromagnetic compatibility requirements described in the following standards:

European Standard EN 45502-2-1

Active Implantable Medical Devices Part 2-1: Particular requirements for active implantable medical devices intended to treat bradyarrhythmia (Cardiac pacemakers).

European Standard prEN 45502-2-2

Active Implantable Medical Devices, Part 2-2: Particular requirements for active implantable medical devices intended to treat tachyarrhythmia (includes implantable defibrillators).

The standards above are based on the EN 50061, Cardiac Pacemakers, Specifications for implantable cardiac pacemakers, Section 6.3, and Electromagnetic Compatibility.

2007 American National Standard ANSI/AAMI PC69, 2<sup>nd</sup> edition  
Implantable Medical Devices – Electromagnetic compatibility.

St. Jude Medical does not provide recommendations regarding a patient's ability to return to work. Technical Services can provide technical assistance to the employer or testing consultant in interpreting test results related to various EMI sources and potential responses.

If you need additional information or would like to discuss this subject further, please call Technical Services at 1-800-722-3774.

## Environmental EMI testing Consultants

DNB Engineering  
Fullerton, CA  
(800) 282-1462  
Doug Broaddus  
WEB: [www.dnbenginc.com](http://www.dnbenginc.com)

Wyle Laboratories  
Huntsville, AL  
256-837-4411  
Bobby Hardy  
WEB: [www.wylelabs.com](http://www.wylelabs.com)

EF Electronics  
Aurora, IL  
630-897-1950  
Ed French  
EMAIL: [efemctest@aol.com](mailto:efemctest@aol.com)

American Industrial Hygiene Association  
Fairfax, VA  
703-849-8888  
WEB: [www.aiha.org](http://www.aiha.org)

Radiometrics Midwest Corporation  
Romeoville, IL  
815-293-0772  
WEB: [www.radiomet.com](http://www.radiomet.com)

Dave Baron, PE  
Consulting Engineer  
Electromagnetic Field Safety  
Austin, TX 512-917-8346  
Minneapolis MN 612-722-4027  
EMAIL: [d.baron@ieee.org](mailto:d.baron@ieee.org)